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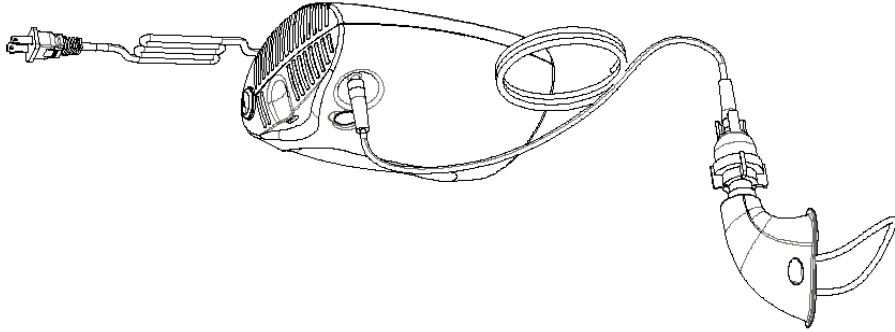
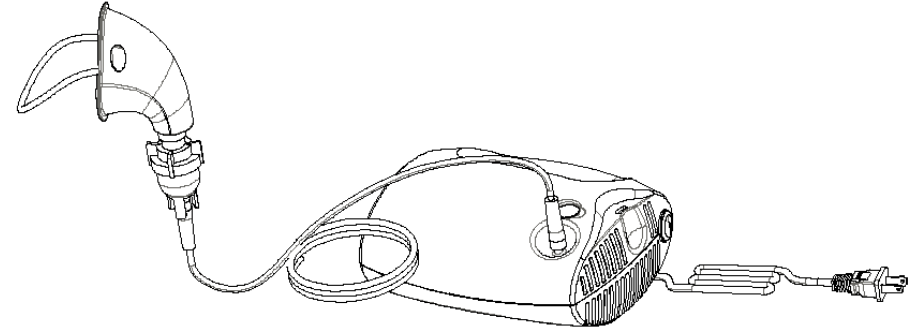
### Models: 1621, 1622, & 1623

CAUTION: Rx ONLY

The Compressor Nebulizers 1621, 1622 and 1623 are intended for use in the treatment of asthma, Chronic Obstructive Pulmonary Disease (COPD) and other respiratory ailments in which an aerosol medication is required during therapy. They suitable for single patient use.

The Compressor Nebulizer 1621, 1622 and 1623 not intended for use with Pentamidine.

# Important Safeguards

This Medi-Pump product is an electrical device. Always follow basic safety precautions when using this device, especially when children are present. The following important information is highlighted by these symbols.

	To signify "CAUTION". Read the instructions carefully before operating device.
	"ON"
○	"OFF"
	"TYPE B" applied part. Note - B = Body
	"DOUBLE INSULATED" Class II Equipment Designation.
	This device bears "CE" mark CE0123 indicating its' conformity with the provision of Council Directive 93/42/EEC concerning medical devices, and fulfills the essential requirement of Annex I of this directive.

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# Important Safeguards

## SAVE THESE INSTRUCTIONS

**DANGER**- To reduce the risk of electrocution:

- 1 Always unplug this product immediately after using.
- 2 Do not use while bathing.
- 3 Do not place or store product where it can fall or be pulled into a tub or sink.
- 4 Do not place in or drop into water or other liquid.
- 5 Do not reach for a product that has fallen into water. Unplug immediately.

**WARNING**- To reduce the risk of burns, electrocution, fire, or injury to persons.

- 1 Never leave this product unattended when plugged in.
  - 2 Use this product only for its intended use as described in the operator manual. Do not use attachments not recommended by the manufacturer.
  - 3 Closely monitor and supervise when used by, on, or near invalids and children.
  - 4 Never operate this product if:
    - a. It has a damaged power cord or plug.
    - b. It is not working properly.
    - c. It has been dropped or damaged.
    - d. It has been dropped into water.
- Return the product to an authorized Medi-Pump ® service center or distributor.
- 5 Keep the power cord away from heated surfaces.
  - 6 Never use while sleeping or drowsy.
  - 7 Never drop or insert any object into any opening or hose.
  - 8 Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.
  - 9 Never block the air openings of the device or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
  - 10 Do not put device closer than 1 inch from wall.

# Table Of Contents

---

INTRODUCTION.....	4
SPECIFICATIONS.....	5
FEATURES.....	6
ACCESSORIES.....	7
OPERATING INSTRUCTIONS.....	8
CLEANING PROCEDURES.....	9
FILTER REPLACEMENT.....	10
TROUBLESHOOTING.....	11
MAINTENANCE.....	12
WORLDWIDE SERVICE CENTERS.....	13
WARRANTY.....	15

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---

**Purpose of Device**

The Compressor Nebulizers 1621, 1622, and 1623 are intended for use in the treatment of asthma, Chronic Obstructive Pulmonary Disease (COPD) and other respiratory ailments in which an aerosol medication is required during therapy. They are suitable for single patient use.

**Description of Device**

This compact, portable and light-weight nebulizer offers convenience while receiving inhalation treatments. Treatments are delivered quickly, safely and conveniently making this unit ideal for all ages.

# Specifications

<b>Model</b>	<b>1621</b>	<b>1622</b>	<b>1623</b>
<b>Power</b>	<b>115V, 60Hz</b>	<b>230V, 50Hz</b>	<b>230V, 60Hz</b>
<b>Power Consumption</b>	<b>80 Watts</b>	<b>65 Watts</b>	<b>80 Watts</b>
<b>Nebulization Rate</b>	<b>0.2~0.3 ml/minute</b>		
<b>Particle Size Diameter</b>	<b>0.5~5µm typical</b>		
<b>Medication Cup Capacity</b>	<b>1~8ml</b>		
<b>Sound Level</b>	<b>≤ 57dB</b>		
<b>Operating Temperature</b>	<b>10 to 40 Degrees C. (50 to 104 Degrees F.)</b>		
<b>Operating Relative Humidity</b>	<b>10% - 95% RH non-condensing</b>		
<b>Operating Atmospheric Pressure Range</b>	<b>700hPa~1060hPa 10 - 15 PSI</b>		
<b>Storage Temperature</b>	<b>5 to 40 Degrees C. (41 to 105 Degrees F.)</b>		
<b>Storage Relative Humidity</b>	<b>10% - 95% RH non-condensing</b>		
<b>Storage Atmospheric Pressure Range</b>	<b>700hPa~1060hPa 10 - 15 PSI</b>		
<b>Dimensions</b>	<b>212×113×178mm</b>		
<b>Weight</b>	<b>1640g</b>		
<b>Driving Gas</b>	<b>Air</b>		
<b>Life Span</b>	<b>3-5 years</b>		
<b>Operating Time</b>	<b>Continuous</b>		

Performance Information provided by this manual in accordance with EN13544-1:2001 may not apply to drugs supplied in suspension or high viscosity form. In such cases, information should be sought from the drug supplier.

- 1) Conditions may vary based on altitude above sea level, barometric pressure and temperature. Specifications subject to change without notice.
- 2) This product complies with the following standards: IEC/EN 60601-1.
- 3) For electromagnetic compatibility the standards are based as follows: IEC/EN 60601-1-2.
- 4) Protection against electric shock is classified as Type B.
- 5) Degree of protection against ingress of water is rated as IPX0.
- 6) Mode of operation is defined as continuous mode.
- 7) Equipment not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.

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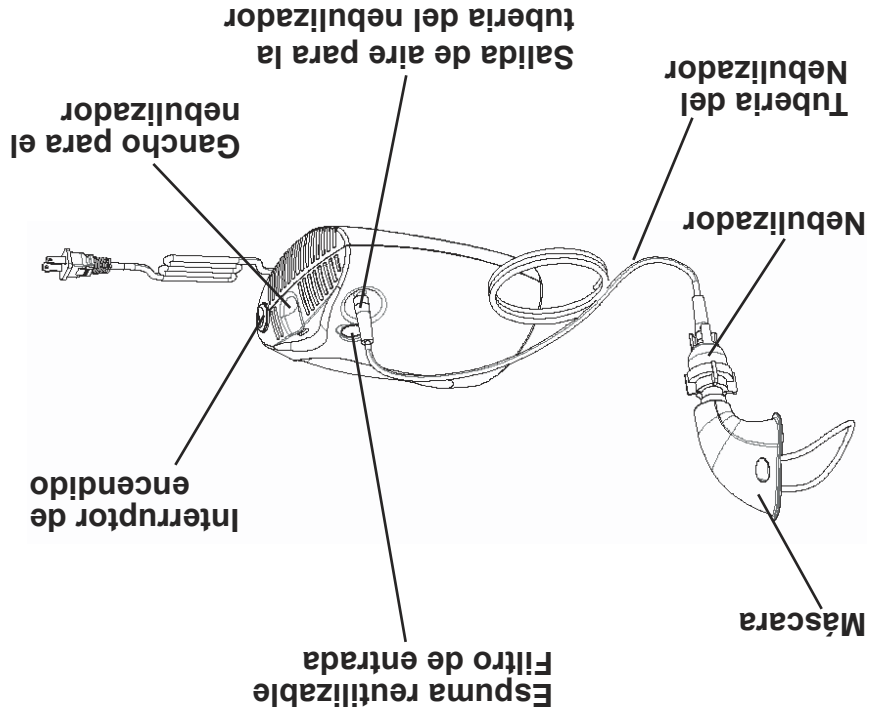
Figura 1

# Important Parts of the Medi-Pump Device

Familiarize yourself with the important parts shown below before set-up.



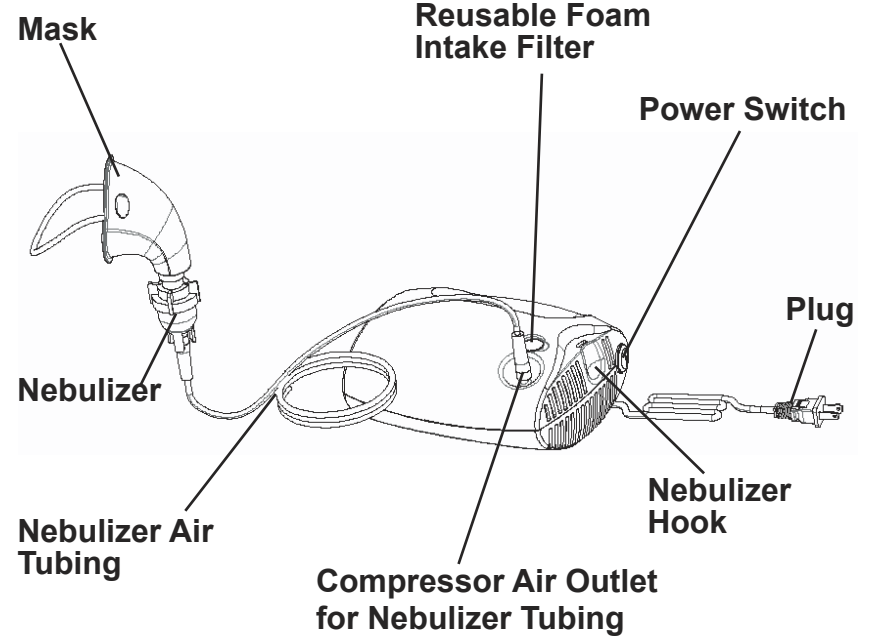
Figure 1
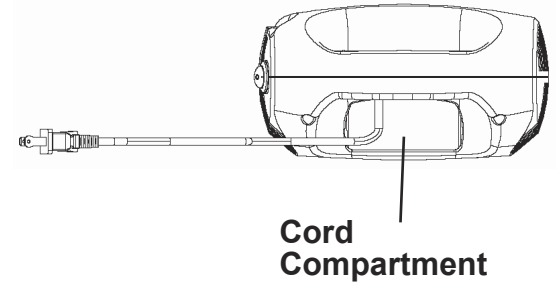
# Accessories

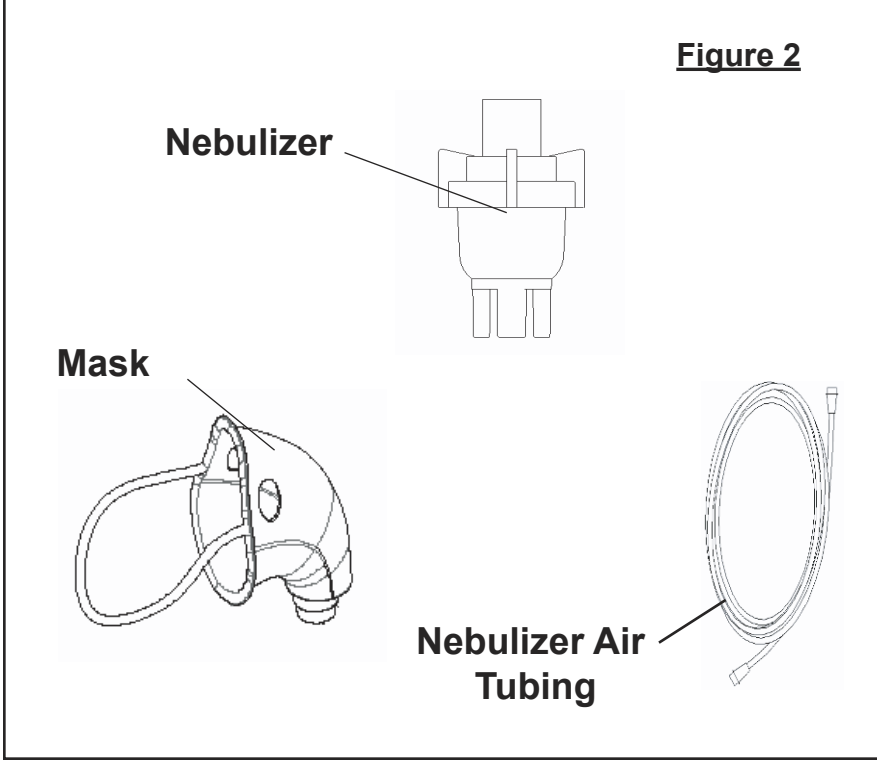The following items are included with your Model 1621, 1622, and 1623 Nebulizer.

The nebulizer is for single use only. The nebulizer should be disposed of in a safe method as dictated by local law or regulation.

Do not use substitutes for these parts. Use Medi-Pump qualified replacement parts only.

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# Operation Instructions

## Operation Instructions

- 1 Plug power cord into electrical outlet.
- 2 Attach one end of plastic tubing to the Nebulizer and the other end to the compressor air outlet.
- 3 Fill Nebulizer with prescribed medication.
- 4 Switch to “ON” to begin treatment.
- 5 Switch to “OFF” when treatment is completed.
- 6 Unplug unit from electrical source. Cord may be coiled and placed in compartment underneath unit.
- 7 Clean Nebulizer and accessories as directed by manufacturer.
- 8 Compressor case may be cleaned with a damp cloth.

**Notice: Unit must be unplugged before cleaning.**

 **Warning**  
Use the medication the way it was intended and as instructed by your physician or qualified health care professional only.

# Cleaning the Device

- 1 Make sure the ON/OFF switch is in the OFF (○) position.
- 2 Unplug the device from the wall outlet.
- 3 Remove Nebulizer, mask and tubing.
- 4 Wash these parts in a hot water/ dishwashing detergent solution and rinse with hot tap water. Soak in a solution of one part vinegar to three parts hot water for 30 minutes. Rinse with hot tap water and air dry completely before using.
- 5 Wipe the outside of the main body with a clean, damp cloth after each use.

**CAUTION**  
Unplug the unit from the wall outlet before cleaning.

**CAUTION**  
Never submerge this device in water, hold under running water or spray with water or other liquids. This will result in damage to the device.

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# Filter Replacement

## Filter Replacement

The filter should be replaced every three months or if contaminated.

- 1 Grasp the filter cover firmly and pull up.
- 2 Lift out the old filter and replace with the new filter.
- 3 Push the filter cover back into the filter fitting.

## Note

If the filter cover is dusty or clogged, hold it under running tap water to clean. Air dry.

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# Troubleshooting

Symptoms	Remedies
Unit does not turn on when Power switch is in "ON" position.	Make sure the AC Power Cord is correctly connected to a power source.
Unit is "ON" but there is no misting, or a low rate of misting.	<ol style="list-style-type: none"> <li>1 Check for medication in the nebulizer</li> <li>2 Check the nebulizer for damage.</li> <li>3 Check that tubing is not bent.</li> <li>4 Check that tubing is attached to nebulizer tightly.</li> </ol>

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# Maintenance

All major maintenance on the device must be performed by an authorized Medi-Pump service center or distributor.

## Caution

This device is oil-less, do not lubricate.

## Note

It will be necessary for you to provide your model number and serial number when contacting a service center for help.

You will find this information on the nameplate located on the bottom of the device. Please take a moment to record this information for future use.

## WARNING

Do not take nebulizer case apart. Failure to comply could result in electrical shock. All disassembly must be carried out by an authorized Medi-Pump service center or distributor.

## BOLIVIA

Importadora Fernando  
Calle Tucabaca Esq. Burapuca  
Casilla No. 5  
Santa Cruz, BOLIVIA  
Phone: (591) (3) 354-2525  
Fax: (591) (3) 354-2526  
Email: imp-fernando@cotas.com.bo  
Website: www.importadorafernando.com.bo

## CHILE

Dipromed S.A.  
Palacio Riesco 4331  
Huechuraba  
Santiago, CHILE  
Phone: (56) (2) 473-1300  
Fax: (56) (2) 473-1399  
Email: informaciones@dipromed.cl  
Website: www.dipromed.cl

## INDURA S.A.

Camino A Melipilla 7060, Cerrillos  
Casilla 13850, Correo 21  
Santiago, CHILE  
Phone: (56) (2) 343-2538  
Fax: (56) (2) 343-2540  
Email: gmendez@indura.net  
Website: www.indura.net

## Tecnigen S.A.

Román Díaz 340, Providencia Casilla 192-55  
Santiago, CHILE  
Phone: (56) (2) 235-8011  
Fax: (56) (2) 235-7385  
Email: gmartinez@tecnigen.cl  
Website: www.tecnigen.cl

## COLOMBIA

Imcolmedica S.A.  
Calle 36 No. 15-42 A.A. 30892  
Santafe De Bogota, D.C., COLOMBIA  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: imcolmedica@imcolmedica.com.co  
Website: www.imcolmedica.com.co

## COSTA RICA

Representaciones G.M.G., S.A.  
Apartado 1110 Centro  
Colón, San José 1007 COSTA RICA  
Phone: (506) 232-1164  
Fax: (506) 232-5615  
Email: repgmsa@racsa.co.cr

## DOMINICAN REPUBLIC

Farmaco Química Nacional  
Avenida José A. Aybar C. Esq  
Clarín, Apartado 1069  
Santo Domingo, DOMINICAN REPUBLIC  
Phone: (80) (9) (6) 539-5545  
Fax: (80) (9) (6) 539-5548  
Email: farmaconal@codetel.net.do  
Website: www.farmaconal.com

## ECUADOR

Bioelectronica Blanco S.A.  
Centro Comercial Urdesa  
Bloque C, Local 12, Datiles Y Tercera  
Guayaquil, ECUADOR  
Phone: (593) (4) 288-1569  
Fax: (593) (4) 288-1331  
Email: blanco@biogrupos.com

## EL SALVADOR

OXGASA  
25 Avenida Norte 1080  
PO Box 1968  
San Salvador, EL SALVADOR  
Phone: (503) (2) 234-3200  
Fax: (503) (2) 226-0302  
Email: oxgasa-gerenciaventas@salnet.net  
Website: www.oxgasa.com

## GUATEMALA

Casa Medica  
12 Calle 3-20, Zona 1  
Guatemala City, GUATEMALA  
Phone: (502) 232-8230  
Fax: (502) 232-8232  
Email: gerencia@casa-medica.com  
Website: www.casa-medica.com

## Importadora Jaeger, S.A.

PO Box 1556  
12 Calle 8-53, Zona 1  
Guatemala City, 01001 GUATEMALA  
Phone: (502) (2) 232-2285  
Fax: (502) (2) 251-4137  
Email: info@jaeger.com.gt  
Website: www.jaeger.com.gt

## MEXICO

ISE  
(Ingeniería y Sistemas Electromedicos)  
Ramón Treviño  
Playa Regatas 3341  
Col. Reforma Iztaccihuatl,  
Esquina Playa Erizo, c.p. 08810  
Mexico, DF - MEXICO  
Phone: (01-81) 8372-0338  
Fax: (01-81) 8374-1170  
Email: contacto@isemexico.com.mx  
Website: www.isemexico.com.mx

## NICARAGUA

Sinter, S.A.  
Rotonda de El Periodista  
100 Metros Al Sur, Continguo A Ofplaza  
Managua, NICARAGUA  
Phone: (50) (5) 278-0177  
Fax: (50) (5) 278-0252  
Email: sinter@ibw.com.ni

## PANAMA

Horacio Icaza y Cia, S.A.  
Avenida Justo Arosemena  
Calle 44 Este Bella Vista  
Panama City, 1 PANAMA  
Phone: (50) (7) 207-6300  
Fax: (50) (7) 227-5682  
Email: trafico.mortega@casamedic.com.pa  
Website: www.casamedic.com.pa

## PERU

A. Jaime Rojas  
Representaciones Generales S.A.  
Jr. Garcia y Garcia No. 870  
Barranco, Lima 4 PERU  
Phone: (51) (1) 477-8410  
Fax: (51) (1) 477-1316  
Email: ventas@ajaimerojas.com  
Website: www.ajaimerojas.com

## PERU

Comindent Tarrillo Barba S.A.C.  
Av. Emancipacion 267  
PO Box No. 3178  
Lima, 1 PERU  
Phone: (51) (1) 428-6429  
Fax: (51) (1) 426-6394  
Email: comindent@infonegocio.net.pe

## TRINIDAD & TOBAGO

A.A. Laquis LTD  
12-16 London Street  
Port of Spain  
Trinidad, West Indies TRINIDAD &  
TOBAGO  
Phone: (868) 624-6995  
Fax: (868) 624-4215  
Email: aalmed@trinidad.net  
Website: www.aalaquis.com

## URUGUAY

Elena Ltda.  
Br. Artigas 1631 Of. 1101  
Montevideo, 11200 URUGUAY  
Phone: (598) (2) 409-0347  
Fax: (598) (2) 400-3570  
Email: asuberb@adinet.com.uy

## VENEZUELA

Tecnomed J. Trapp C.A.  
Peligro a Puente República  
Edif. República, Piso 1, Av. Este 2  
La Candelaria, Caracas, VENEZUELA  
Phone: (582) (12) 575-2422  
Fax: (582) (12) 576-2444  
Email: tecnomed@cantv.net  
Website: www.tecnomedjtrapp.com

PERU  
Comindent Tarrillo Barba S.A.C.  
Av. Emancipacion 267  
PO Box No. 3178  
Lima, 1 PERU  
Phone: (51) (1) 428-6429  
Fax: (51) (1) 426-9394  
Email: comindent@infonegocio.net.pe  
Website: www.oxgasa.com

GUATEMALA & TOBAGO  
A.A. Laquis LTD  
12-16 London Street  
Port of Spain  
Trinidad, West Indies TRINIDAD &  
TOBAGO  
Phone: (868) 624-6995  
Fax: (868) 624-4215  
Email: aalmed@trinidad.net

URUGUAY  
Elena Ltda.  
Br. Artigas 1631 Of. 1101  
Montevideo, 11200 URUGUAY  
Phone: (598) (2) 409-0347  
Fax: (598) (2) 400-3570  
Email: asuberb@adinet.com.uy

VENEZUELA  
Tecnomed J. Trapp C.A.  
Peligro a Puente República  
Edif. República, Piso 1, Av. Este 2  
La Candelaria, Caracas, VENEZUELA  
Phone: (582) (12) 575-2422  
Fax: (582) (12) 576-2444  
Email: tecnomed@cantv.net  
Website: www.tecnomedjtrapp.com

MEXICO  
ISE  
(Ingeniería y Sistemas Electromedicos)  
Ramón Treviño  
Playa Regatas 3341  
Col. Reforma Iztaccihuatl,  
Esquina Playa Erizo, c.p. 08810  
Mexico, DF - MEXICO  
Phone: (01-81) 8372-0338  
Fax: (01-81) 8374-1170  
Email: contacto@isemexico.com.mx  
Website: www.isemexico.com.mx

NICARAGUA  
Sinter, S.A.  
Rotonda de El Periodista  
100 Metros Al Sur, Continguo A Ofplaza  
Managua, NICARAGUA  
Phone: (50) (5) 278-0177  
Fax: (50) (5) 278-0252  
Email: sinter@ibw.com.ni

PANAMA  
Horacio Icaza y Cia, S.A.  
Avenida Justo Arosemena  
Calle 44 Este Bella Vista  
Panama City, 1 PANAMA  
Phone: (50) (7) 207-6300  
Fax: (50) (7) 227-5682  
Email: trafico.mortega@casamedic.com.pa  
Website: www.casamedic.com.pa

CHILE  
Dipromed S.A.  
Palacio Riesco 4331  
Huechuraba  
Santiago, CHILE  
Phone: (56) (2) 473-1300  
Fax: (56) (2) 473-1399  
Email: informaciones@dipromed.cl  
Website: www.dipromed.cl

INDURA S.A.  
Camino A Melipilla 7060, Cerrillos  
Casilla 13850, Correo 21  
Santiago, CHILE  
Phone: (56) (2) 343-2538  
Fax: (56) (2) 343-2540  
Email: gmendez@indura.net  
Website: www.indura.net

Tecnigen S.A.  
Román Díaz 340, Providencia Casilla 192-55  
Santiago, CHILE  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: imcolmedica@imcolmedica.com.co  
Website: www.imcolmedica.com.co

COLOMBIA  
Imcolmedica S.A.  
Calle 36 No. 15-42 A.A. 30892  
Santafe De Bogota, D.C., COLOMBIA  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: gmartinez@tecnigen.cl  
Website: www.tecnigen.cl

COSTA RICA  
Representaciones G.M.G., S.A.  
Apartado 1110 Centro  
Colón, San José 1007 COSTA RICA  
Phone: (506) 232-1164  
Fax: (506) 232-5615  
Email: repgmsa@racsa.co.cr

DOMINICAN REPUBLIC  
Farmaco Química Nacional  
Avenida José A. Aybar C. Esq  
Clarín, Apartado 1069  
Santo Domingo, DOMINICAN REPUBLIC  
Phone: (80) (9) (6) 539-5545  
Fax: (80) (9) (6) 539-5548  
Email: farmaconal@codetel.net.do  
Website: www.farmaconal.com

ECUADOR  
Bioelectronica Blanco S.A.  
Centro Comercial Urdesa  
Bloque C, Local 12, Datiles Y Tercera  
Guayaquil, ECUADOR  
Phone: (593) (4) 288-1569  
Fax: (593) (4) 288-1331  
Email: blanco@biogrupos.com

EL SALVADOR  
OXGASA  
25 Avenida Norte 1080  
Calle Tucabaca Esq. Burapuca  
Casilla No. 5  
Santa Cruz, BOLIVIA  
Phone: (591) (3) 354-2525  
Fax: (591) (3) 354-2526  
Email: imp-fernando@cotas.com.bo  
Website: www.importadorafernando.com.bo

CHILE  
Dipromed S.A.  
Palacio Riesco 4331  
Huechuraba  
Santiago, CHILE  
Phone: (56) (2) 473-1300  
Fax: (56) (2) 473-1399  
Email: informaciones@dipromed.cl  
Website: www.dipromed.cl

INDURA S.A.  
Camino A Melipilla 7060, Cerrillos  
Casilla 13850, Correo 21  
Santiago, CHILE  
Phone: (56) (2) 343-2538  
Fax: (56) (2) 343-2540  
Email: gmendez@indura.net  
Website: www.indura.net

Tecnigen S.A.  
Román Díaz 340, Providencia Casilla 192-55  
Santiago, CHILE  
Phone: (56) (2) 235-8011  
Fax: (56) (2) 235-7385  
Email: gmartinez@tecnigen.cl  
Website: www.tecnigen.cl

COLOMBIA  
Imcolmedica S.A.  
Calle 36 No. 15-42 A.A. 30892  
Santafe De Bogota, D.C., COLOMBIA  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: imcolmedica@imcolmedica.com.co  
Website: www.imcolmedica.com.co

COSTA RICA  
Representaciones G.M.G., S.A.  
Apartado 1110 Centro  
Colón, San José 1007 COSTA RICA  
Phone: (506) 232-1164  
Fax: (506) 232-5615  
Email: repgmsa@racsa.co.cr  
Website: www.farmaconal@codetel.net.do

ECUADOR  
Bioelectronica Blanco S.A.  
Centro Comercial Urdesa  
Bloque C, Local 12, Datiles Y Tercera  
Guayaquil, ECUADOR  
Phone: (591) (1) 477-8410  
Fax: (51) (1) 477-1316  
Email: ventas@ajaimerojas.com  
Website: www.ajaimerojas.com

PERU  
Comindent Tarrillo Barba S.A.C.  
Av. Emancipacion 267  
PO Box No. 3178  
Lima, 1 PERU  
Phone: (51) (1) 428-6429  
Fax: (51) (1) 426-9394  
Email: comindent@infonegocio.net.pe  
Website: www.oxgasa.com

GUATEMALA & TOBAGO  
A.A. Laquis LTD  
12-16 London Street  
Port of Spain  
Trinidad, West Indies TRINIDAD &  
TOBAGO  
Phone: (868) 624-6995  
Fax: (868) 624-4215  
Email: aalmed@trinidad.net

URUGUAY  
Elena Ltda.  
Br. Artigas 1631 Of. 1101  
Montevideo, 11200 URUGUAY  
Phone: (598) (2) 409-0347  
Fax: (598) (2) 400-3570  
Email: asuberb@adinet.com.uy

VENEZUELA  
Tecnomed J. Trapp C.A.  
Peligro a Puente República  
Edif. República, Piso 1, Av. Este 2  
La Candelaria, Caracas, VENEZUELA  
Phone: (582) (12) 575-2422  
Fax: (582) (12) 576-2444  
Email: tecnomed@cantv.net  
Website: www.tecnomedjtrapp.com

MEXICO  
ISE  
(Ingeniería y Sistemas Electromedicos)  
Ramón Treviño  
Playa Regatas 3341  
Col. Reforma Iztaccihuatl,  
Esquina Playa Erizo, c.p. 08810  
Mexico, DF - MEXICO  
Phone: (01-81) 8372-0338  
Fax: (01-81) 8374-1170  
Email: contacto@isemexico.com.mx  
Website: www.isemexico.com.mx

NICARAGUA  
Sinter, S.A.  
Rotonda de El Periodista  
100 Metros Al Sur, Continguo A Ofplaza  
Managua, NICARAGUA  
Phone: (50) (5) 278-0177  
Fax: (50) (5) 278-0252  
Email: sinter@ibw.com.ni

PANAMA  
Horacio Icaza y Cia, S.A.  
Avenida Justo Arosemena  
Calle 44 Este Bella Vista  
Panama City, 1 PANAMA  
Phone: (50) (7) 207-6300  
Fax: (50) (7) 227-5682  
Email: trafico.mortega@casamedic.com.pa  
Website: www.casamedic.com.pa

CHILE  
Dipromed S.A.  
Palacio Riesco 4331  
Huechuraba  
Santiago, CHILE  
Phone: (56) (2) 473-1300  
Fax: (56) (2) 473-1399  
Email: informaciones@dipromed.cl  
Website: www.dipromed.cl

INDURA S.A.  
Camino A Melipilla 7060, Cerrillos  
Casilla 13850, Correo 21  
Santiago, CHILE  
Phone: (56) (2) 343-2538  
Fax: (56) (2) 343-2540  
Email: gmendez@indura.net  
Website: www.indura.net

Tecnigen S.A.  
Román Díaz 340, Providencia Casilla 192-55  
Santiago, CHILE  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: imcolmedica@imcolmedica.com.co  
Website: www.imcolmedica.com.co

COLOMBIA  
Imcolmedica S.A.  
Calle 36 No. 15-42 A.A. 30892  
Santafe De Bogota, D.C., COLOMBIA  
Phone: (57) (1) 287-1811  
Fax: (57) (1) 285-4663  
Email: imcolmedica@imcolmedica.com.co  
Website: www.imcolmedica.com.co

COSTA RICA  
Representaciones G.M.G., S.A.  
Apartado 1110 Centro  
Colón, San José 1007 COSTA RICA  
Phone: (506) 232-1164  
Fax: (506) 232-5615  
Email: repgmsa@racsa.co.cr  
Website: www.farmaconal@codetel.net.do

ECUADOR  
Bioelectronica Blanco S.A.  
Centro Comercial Urdesa  
Bloque C, Local 12, Datiles Y Tercera  
Guayaquil, ECUADOR  
Phone: (591) (3) 354-2525  
Fax: (591) (3) 354-2526  
Email: imp-fernando@cotas.com.bo  
Website: www.importadorafernando.com.bo

# Warranty Information

Gardner Denver Thomas Inc. warrants the device for 36 months from the date of manufacture due to faulty parts or workmanship. Check with your dealer for more details.

This warranty is limited to the original purchaser of the device. Any defective part(s) or assembly will be repaired or replaced, at the sole discretion and determination of Thomas Industries Inc. if the unit has not been misused or tampered with during the warranty period.

Normal maintenance items, as outlined in this manual and disposable components are not covered by this warranty. Shipping charges, if any, shall be paid by the purchaser.

*A Gardner Denver Product*

1419 Illinois Avenue  
Sheboygan, WI USA  
Phone: (920) 457-0463  
Fax: (920) 451-6307  
www.rtpumps.com

## Note

You may determine the date of manufacture by reading the first 4 numbers from the serial number field of the nameplate. These numbers represent the month and year that the device was made. For example, if it reads "0204", the date of manufacture is February of 2004. If it reads "1295" the date of manufacture is December of 1995.

## Note

There is no other express warranty. Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensation for costs incurred for compressor rental, or for labor costs incurred in repairing or replacing a defective part(s).

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